# DRUG NOSTIC

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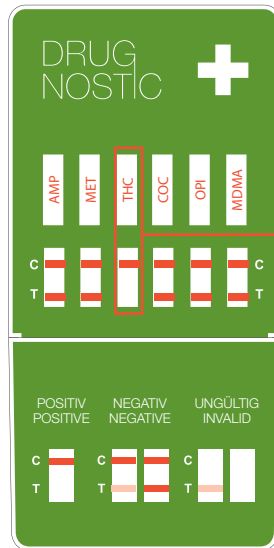
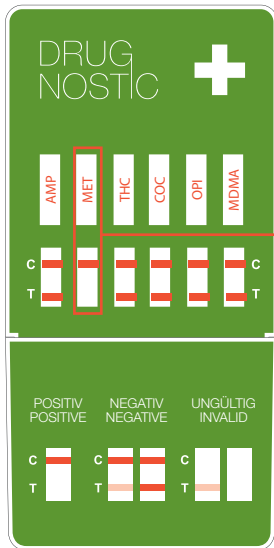
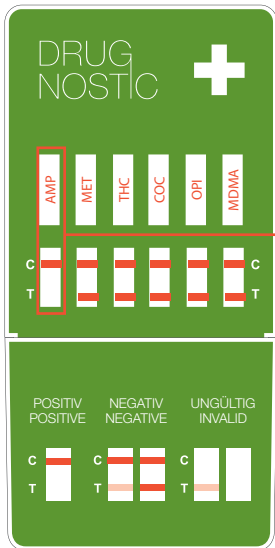
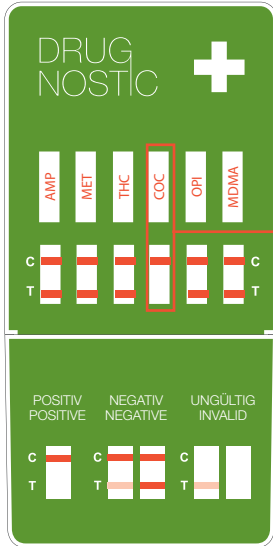
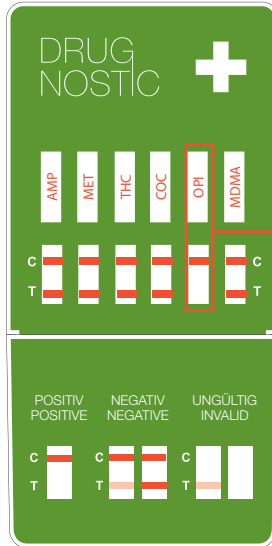
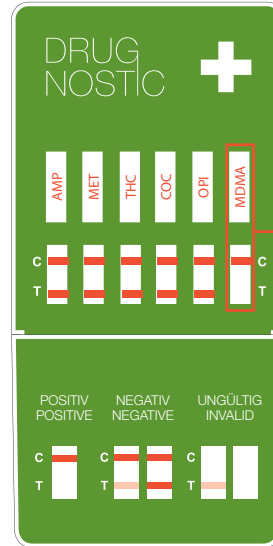
---

Dear customer!

You can use the Drugnostic rapid drug test safely, simply and quickly. The instructions describe how a wide variety of substances can be screened using a modern test method. This test method is used by the police for testing.

With Drugnostic you can examine ...

- 1 solid substances such as tablets, hashish, parts of marijuana plants and powder
- 2 liquids from ampoules or other containers that may contain suspicious substances
- 3 minimal traces of drugs adhering to surfaces such as furniture, utilitarian objects etc. as residues
- 4 urine samples, which can be used to detect drug use

### Contents of pack

- 1 instructions for use
- 2 shrink-wrapped test cassettes
- 1 disposable pipette per test cassette
- 1 bottle of mobile phase

You will also need a watch, disposable gloves and, for urine samples, a test beaker.

In everyday life everyone is confronted with legal drugs such as alcohol and cigarettes. Unfortunately, young people in particular are more and more often getting their experiences with illegal substances such as narcotics. And what starts with inquisitive experimentation often ends in addiction or dependency.

Everyone risks becoming dependent. Many people cite as reasons insecurity and inability to handle conflict, as well as peer pressure, adapting to friends or escaping from everyday problems.

The boundaries between experimentation and addiction overlap. The distinctive features can vary a lot.

## IMPORTANT INFORMATION

---

There are clues that indicate regular use of narcotics.

They include changes in behaviour. A person taking drugs can become aggressive or apathetic. Performance at school often suffers, and it is not uncommon for work to be affected.

Conspicuous lack of interest in one's surroundings and neglect of one's own outward appearance can also be signs of drug use.

In the event of suspicion the Drugnostic rapid test can provide certainty as to whether drug use may actually be the cause.

Nobody can watch their child or partner 24 hours a day to establish or prevent drug use. Any such overprotection will often have an opposite effect to the one you want, and the other person will feel observed. It is far more important to provide children with protective mechanisms for life. With a partner the existing trusting relationship should be expanded on and consolidated.

## HOW CAN DRUGNOSTIC HELP?

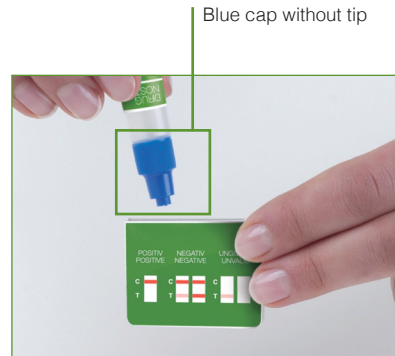
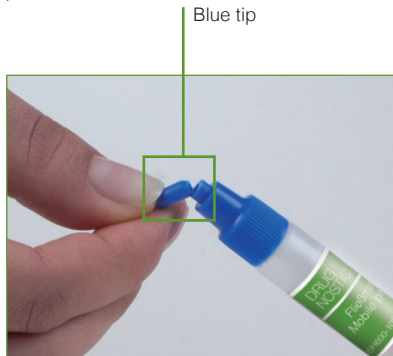
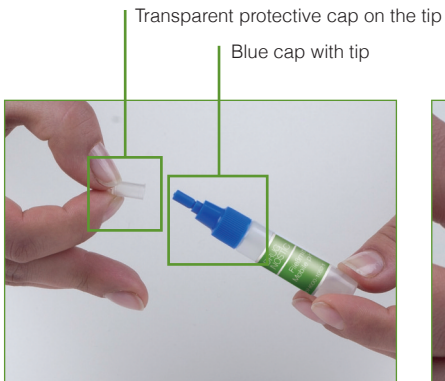
---

Ideally, discuss the matter and then do the test together. It has been shown that Drugnostic, the rapid test, can provide clarity as to whether your child or partner is actually taking drugs.

Drugnostic could also be used as for self-testing, e.g. before a job interview.  
If necessary the Drugnostic test can be repeated at any time to detect consistent drug-abstinence behaviour.


Parents or partners of addicts can also get further advice and help in addiction advice centres, through the Federal Centre for Health Education or the German Paediatricians' Association or via your own local country Drug help centres.

## IMPORTANT INFORMATION – CONTENTS OF PACK


## IMPORTANT INFORMATION – CONTENTS OF PACK

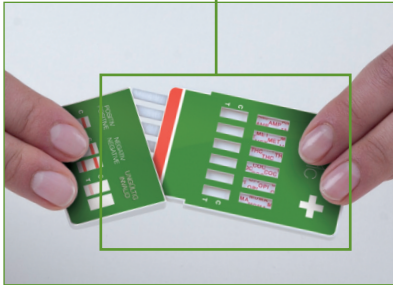
Test

Test strip

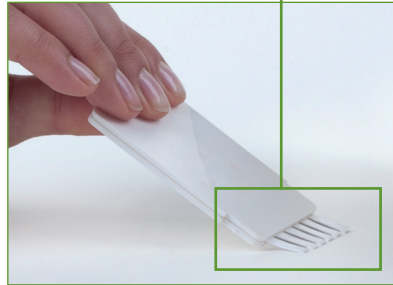Test cap
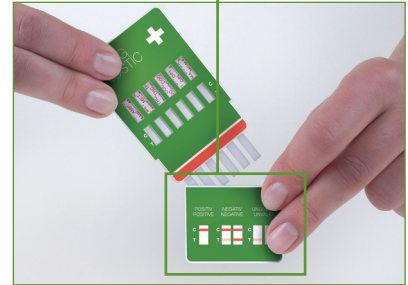
## SCREENING FOR SOLID SUBSTANCES 1

---

Drugnostic, the rapid drug test that screens for solid substances such as tablets, hashish, parts of marijuana plants and powder.

Anyone with children will perhaps recognise the following situation: There are substances in a desk or ashtray that cannot be classified. Traces of a white powder on a plate is cause for concern.

To obtain certainty as to whether your child or partner is taking drugs, discussion is the first step.

Is this rejected? Then Drugnostic, the rapid test, can provide certainty. Following discussion in confidence the test can also form a basis as to how to continue.

### Use:

Using Drugnastic, minute grains or dark crumbs of a solid substance suffice for reliable proof of whether it is a drug.

### Important:

Only examine minute quantities. Tablets, hashish and parts of marijuana plants must be crushed so as to separate off a piece about as big as a pin head as a sample. Small quantities also suffice to test a powder.

## TEST PREPARATION – SOLID SUBSTANCES 1A

Take the bottle of mobile phase out of the pack. Unscrew the blue cap. Put the substance to be tested in the bottle and close it again. Leave it standing (not horizontal) at room temperature overnight, or for about 12 hours.

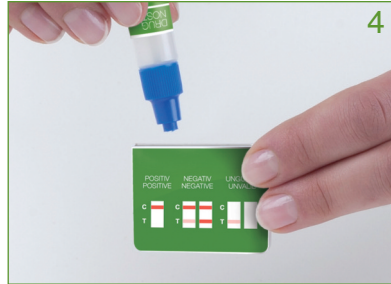
The following day gently shake the bottle with contents so that everything gets mixed together well. Then remove the test cassette from the foil covering (the foil and drying agent can be disposed of with the domestic waste). Remove the cap from the test cassette. Place the cassette on a clean, dry surface.

## TEST PREPARATION – SOLID SUBSTANCES 1A


Pull off the transparent protective cap, and now break off the blue tip.

Carefully pour the total contents of the bottle into the test cap.

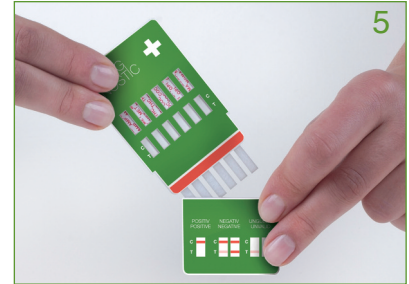NB: Keeping the latter horizontal will prevent the liquid from spilling.

Slowly and carefully put the test into the test cap, test strip first. The whole test must be held vertically so no liquid escapes. Wait for 5 to 8 minutes, as the reaction time. Important: This time must be exactly adhered to, to ensure a correct result. You can find out your personal test result in the Results section.

## SCREENING OF FLUIDS 2

---

Instructions for use of Drugnostic to screen liquids from ampoules or other containers that contain suspicious substances

NB:

The Drugnostic rapid drug test can detect drugs in liquids. The party drug 'Liquid Ecstasy' (GHB – gamma-hydroxybutyrate) and LSD are exceptions to this. They cannot be detected using Drugnostic, as they are synthetic substances.

Many drugs also exist as mixed variants that are available as liquids. This includes so-called hashish oils, LSD and Liquid Ecstasy.

Are you unsure as to whether a bottle or ampoule you have found contains such as substance?

If possible, try to discuss the matter with the owner in confidence, in order to gain clarity. Drugnostic, the rapid drug test, can then provide certainty as to whether one of the drugs (see Results – Test Evaluation) is contained above a certain concentration in the liquid found.

**Use:**

Before using Drugnostic to test liquids, put on a pair of gloves.

You will thereby avoid hazardous substances coming into contact with your skin. Contact with your mouth or eyes must also be avoided. When using, take care not to spill any of it. It is vital that food that has come into contact with suspicious liquids be disposed of.

**Important:**

A positive test result verifies the presence of a specific drug in the carrier liquid. Whether the owner on whom it was found has actually take drugs can only be clarified using a urine sample. For this purpose read the section on use to screen for drugs in urine.

## TEST PREPARATION – LIQUIDS 2A

Take the bottle of mobile phase out of the pack. Unscrew the blue cap. Pour one to three drops of the suspicious liquid into the opened bottle. Screw the cap back on and shake the bottle.

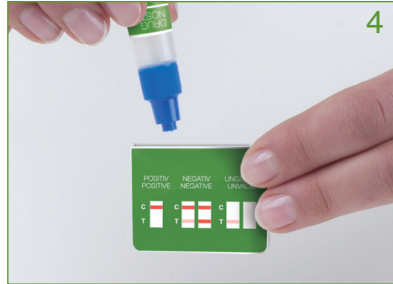
Remove the foil covering from the test cassette (the foil and the drying agent can be disposed of in the household waste). Remove the cap from the test sample. Place the cassette on a clean, dry surface.

## TEST PREPARATION - LIQUIDS 2A


Pull off the transparent protective cap and now break off the blue tip.

Carefully pour the total contents of the bottle into the test cap.

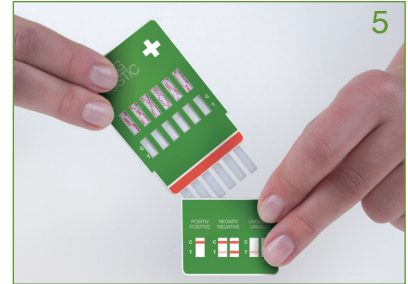NB: Keeping the latter horizontal will prevent spillage of the liquid.

Slowly and carefully put the test into the test cap, test strip first. The whole test must be held vertically so no liquid escapes. Wait for 5 to 8 minutes, as the reaction time. Important: This time must be exactly adhered to, to ensure a correct result. You can find out your personal test result in the Results section.

## SCREENING ON SURFACES 3

---

Instructions for use of Drugnostic to screen for minute drug residues adhering to surfaces such as table tops, cabinets etc.

Anyone suspecting that their child or partner may be taking drugs will be alarmed and will look for signs to verify their suspicions. Grounds for suspicion are created by substances that cannot initially be classified, e.g. traces of a white powder on a table top.

If the other person is careful about not leaving any traces and about keeping their drug use secret it will be even harder to verify.

To find out whether your child or partner is taking drugs, discussion of the matter is always the first step. If this option is refused, and if provision of a urine sample is also refused, the rapid test may bring clarity.

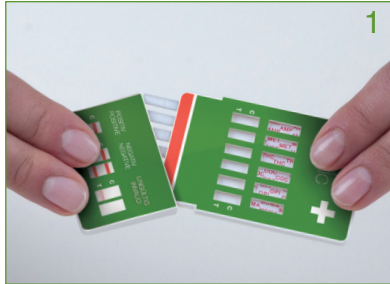
### Use:

Even traces of a drug not visible to the naked eye can be detected using Drugnostic, the rapid drug test. Select for testing those surfaces that are most likely to have come into contact with the drug, i.e. table tops, ashtrays and small mirrors left lying around. Traces of drugs will also be found on surfaces that are above all regularly touched by the test subject, i.e. door handles, school-bag straps, car steering wheels and TV remote controls.


### Important:

A positive test result verifies the presence of a drug on the surface. Whether the drug was actually taken can only be clarified by means of a urine sample. For this purpose read the instructions on screening for drugs in urine.

## TEST PREPARATION – SURFACES 3A

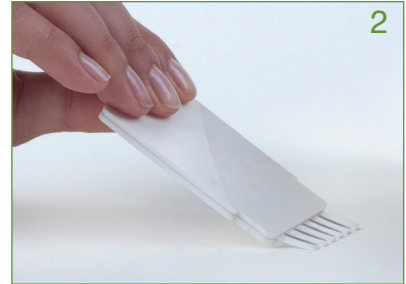Remove the foil covering from the test cassette (the foil and the drying agent can be disposed of in the household waste). Remove the cap from the test sample.


Wipe over one or more suspicious surfaces with the free end of the test strip. **IMPORTANT:** The back of the test cassette must be facing upwards, so the rough side of the test strip touches the surface.


## TEST PREPARATION – SURFACES 3A

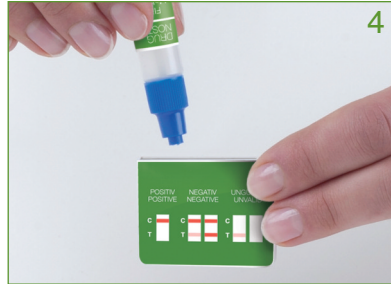Pull off the transparent protective cap and now break off the blue tip.

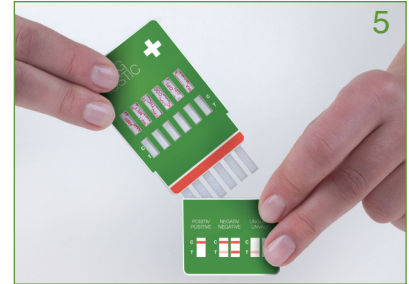Carefully pour the total contents of the bottle into the test cap.  
NB: Keeping the latter horizontal will prevent spillage of the sample liquid.

Slowly and carefully put the test into the test cap, test strip first. The whole test must be held vertically so no liquid escapes. Wait for 5 to 8 minutes, as the reaction time. Important: This time must be exactly adhered to, to ensure a correct result. You can find out your personal test result in the Results section.

## SCREENING FOR DRUGS IN URINE 4 + 4A

### Instructions for use of Drugnostic to screen for drugs in urine samples

In a urine sample the presence of drugs can be verified beyond doubt. It is thus the perfect material for screening using the Drugnostic rapid drug test.

To prevent manipulation of a urine sample please make sure there is no deception in the provision of the sample. No other liquids must be poured into the test beaker. Contamination with cleaning agents that can falsify the test result must be prevented.

Everyone involved in the test should wear gloves, to avoid contact with the urine sample.

Remove the foil covering from the test cassette (the foil and the drying agent can be disposed of in the household waste). Open the cap of the test cassette. The area for reception of the sample is free. Place the cassette on a clean, dry surface.

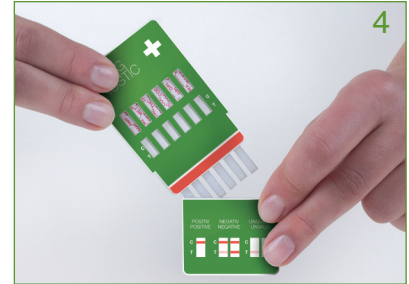
## TEST PREPARATION - URINE 4A

Using the pipette, collect urine from the urine beaker: to do so, **slowly** squeeze the little pipette bulb, to prevent air being sucked into the pipette. Dip the tip with the opening into the urine, **keeping it straight**. Reduce the pressure on the bulb. Urine will now be sucked into the pipette.

Once the pipette is full of urine, the urine can be transferred into the cap. Transfer the entire contents of the pipette into the cap of the test cassette. **Repeat this process four times**. NB: **Keep the cassette horizontal on an even surface, to avoid urine spillage**.


Slowly and carefully put the test into the test cap, test strip first. The entire test must be kept vertical, to avoid any urine escaping. **Wait for 5 to 8 minutes, as the reaction time**. Important: **This time must be exactly adhered to, to ensure a correct result**. You can find out your personal test result in the Results section.

## SCREENING FOR DRUGS IN URINE 4B

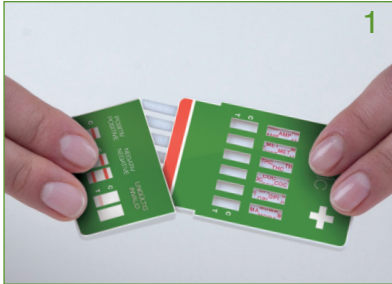---

Instructions, see p. 22


## TEST PREPARATION - URINE 4B

Remove the foil covering from the test cassette (the foil and the drying agent can be disposed of in the household waste). Open the cap of the test cassette. The area for reception of the sample is free. Place the cassette on a clean, dry surface.

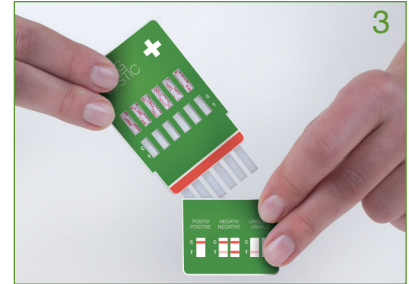For direct collection of the sample without a pipette, dip the test strip vertically into the urine for at least 10-15 seconds. **Make sure the red line does not come into contact with the urine.**

Put the test into the test cap, test strip first. Wait for 5 to 8 minutes, as the reaction time. **Important: This time must be exactly adhered to, to ensure a correct result.** You can find out your personal test result in the Results section.

## TEST EVALUATION

Dear customer,

The reaction time for this rapid drug test is just 5 to 8 minutes. Throughout this period the test cassette must remain in the cap. Then read off your result from the test strip.

After 15 minutes the result can no longer be evaluated.

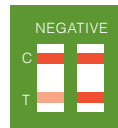
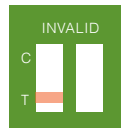
Taking a reading:

The coloured upper section of the test cassette has six windows, which show the names of the drugs that can be evaluated using the test. The six windows below this, marked C and T, are the actual read-out windows.

A red line must appear on the C-line (control line), indicating that the test has functioned without error.

Your personal test result for each drug will be displayed on the T-line (test line). If it is positive, i.e. a drug has been detected beyond doubt, there will be no line on T. In the event of a negative test result (i.e. no drug has been detected beyond doubt) a reddish stroke will appear in the read-out window.

C = control line | T = test line

# DRUG NOSTIC

AMP

MET

THC

COC

OPI

MDMA

C

T

C

T

POSITIV  
POSITIVE

NEGATIV  
NEGATIVE

UNGÜLTIG  
INVALID

C

T

C

T

C

T

C

T

Drug name

Read-out window Result

Key Result

## ABBREVIATIONS DRUG NAMES

AMP

amphetamine (e.g. speed ...) 1000 ng/ml

MET

methamphetamine (e.g. pep ...) 1000 ng/ml

THC

marijuana, hashish (tetrahydrocannabinol) 50 ng/ml

COC

cocaine (e.g. coke, crack ...) 300 ng/ml

OPI

opium - morphine (e.g. heroin, opiates, codeine ...) 2000 ng/ml

MDMA

ecstasy 500 ng/ml

## RESULTS – TEST EVALUATION

Important:

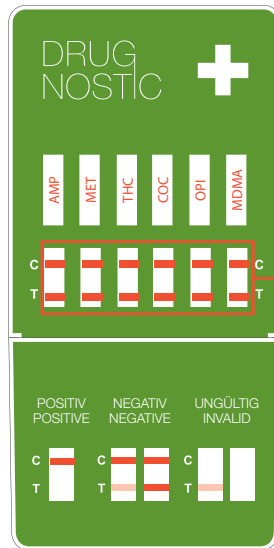
There can be great variation in the intensity of the C- and T-lines.

C = control line

Shows that the test is meaningful – it has been run.

T = test line

Even **very weak reddish lines** are to be evaluated, because in each instance it is the quantity of the drug detected that is important (as well as the time of the last intake, the frequency of intake and the concentration/purity of the drug).

Test is negative

C- and T-lines appear.  
No drugs have been detected.

DRUG NOSTIC

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C T

Test is positive  
Only C-lines appear.  
Drugs have been detected

POSITIV POSITIVE	NEGATIV NEGATIVE	UNGÜLTIG INVALID
C: [red line] T: [no line]	C: [red line] T: [red line]	C: [red line] T: [no line]

DRUG NOSTIC

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C T

Test is invalid  
Only T-lines appear. The test must be repeated

POSITIV POSITIVE	NEGATIV NEGATIVE	UNGÜLTIG INVALID
C: [red line] T: [no line]	C: [red line] T: [red line]	C: [red line] T: [no line]

DRUG NOSTIC

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C T

Test is invalid  
No lines appear. The test must be repeated.

POSITIV POSITIVE	NEGATIV NEGATIVE	UNGÜLTIG INVALID
C: [red line] T: [no line]	C: [red line] T: [red line]	C: [red line] T: [no line]

# RESULTS – TEST EVALUATION

DRUG NOSTIC +

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C T C T C T C T

POSITIV POSITIVE NEGATIV NEGATIVE UNGÜLTIG INVALID

C T C T C T C T

AMP positive

Amphetamine has been detected

DRUG NOSTIC +

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C T C T C T C T

POSITIV POSITIVE NEGATIV NEGATIVE UNGÜLTIG INVALID

C T C T C T C T

MET positive

Methamphetamine has been detected

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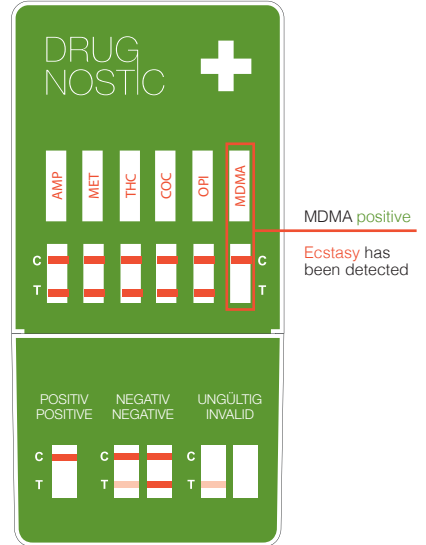
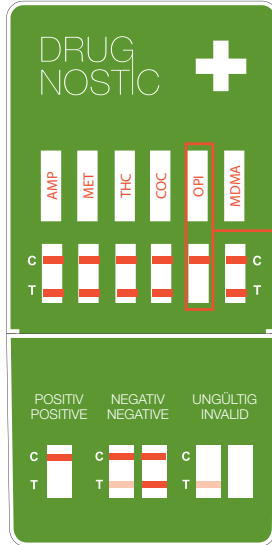
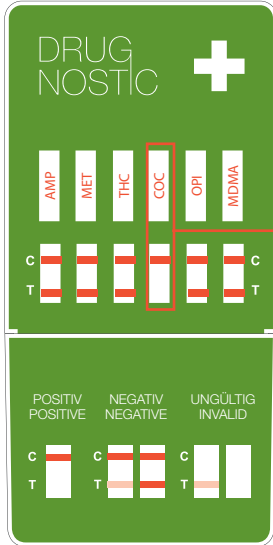
## CONTENTS

2 test cassettes, each with 1 disposable pipette, 1 bottle of  
mobile phase, 1 set of instructions for use. You will also need:  
a watch and disposable gloves

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Gecko Pharma Vertrieb GmbH