

# Home Drug Test

Rapid screen of Urine Samples for  
amphetamines, cocaine,  
methamphetamine, opiates (heroin),  
marijuana and MDMA (ecstasy)

## Instructions for use



**SELFSCREEN®**

## Cut off Limits

Amphetamine	1,000 ng / ml
Cocaine	300 ng / ml
Methamphetamine	1,000 ng / ml
Opiates	2,000 ng / ml
THC – Marijuana	50 ng / ml
MDMA – Ecstasy	500 ng / ml

## Introduction

Withdrawal from family life, a constant need for money, poor grades in school and an increase in aggressive behaviour are not necessarily signs of drug addiction, especially in youth going through puberty. However, warning signals should go off if there is a change in a person's circle of friends and/or excessive partying on a regular basis, e.g. if the test person regularly stays out all night and only comes home in the morning.

Try to persuade the person in question to voluntarily provide you with a urine sample. Many drug users come up with extraordinary ideas on how to manipulate urine samples.

At times, these ideas are passed on or made public on the Internet to help drug users avoid being caught by the now EU-wide traffic controls of the police. If a urine sample is provided, please make sure that no liquids are accessible in the room where the sample is taken, which could be used as a replacement for the urine in the container.

Make sure that no other person is allowed in the room and check also for scents such as citrus fragrance or chlorine, which indicate the adding of cleaning agents to the sample. These are sometimes added to the samples to disrupt drug detection through chemical reaction.

Request that the person involved provide a urine sample in the container provided. It is recommended you wear gloves for hygiene reasons.

Please read the directions carefully, as they contain important information on how to use the **SELFSCREEN**<sup>®</sup> drug screening test.

The test system is an analytical procedure used to test Urine Samples for Drugs

## Description of Tests

<b>AMP</b>	Amphetamine (e.g. speed,...)
<b>COC</b>	Cocaine (e.g. cocaine)
<b>MET</b>	Methamphetamine (e.g. pep,...)
<b>OPI</b>	Opiates (e.g. heroin, morphine...)
<b>THC</b>	Tetrahydrocannabinol (e.g. hashish, marijuana, cannabis)
<b>MDMA</b>	Ecstasy

### Disclaimer

**SELFSCREEN®** Home Drug Test is a highly accurate screen for drugs of abuse in Urine and a positive result is an indicator of the likely presence of a Drug of abuse.

False test results may be caused by the intake of some prescribed or over the counter medication (Nurofen plus, for example, may produce a false positive Opiate result).

**Test results should be verified through your Doctor, by an appropriate laboratory test (HPLC/GCMS) prior to acting on results.**

## Test Procedure



Remove end cap from  
**SELFSCREEN**® cassette



- Dip Cassette Strips into Urine Sample approx half way for 15 seconds, Do not allow the cassette body to touch the specimen.
- Withdraw and replace Cassette cap
- Place cassette on level surface for **5 to 8** minutes and then read results

**DO NOT INTERPRET  
AFTER 10 MINUTES**


## Reading Test Results

The red lines at the height of the “C” in the reading window must appear for the test to be valid. The “C” lines confirm that the test functioned without any errors. If the “C” lines do not appear, then the test is invalid and must be repeated (see page 7, Figures 9 and 10). The “T” line is the test line and indicates the results for each separate drug.

In the event of a negative test result for all drugs tested then a red line will appear at the height of the “T” in all six reading windows (see page 7, Figure 7).

In the event of a positive test result (i.e. a drug was detected) no line will appear at the height of the “T” in the relevant window (see page 7, Figure 8).

**A line, even if faint, in any “T” window should be regarded as a negative result.**

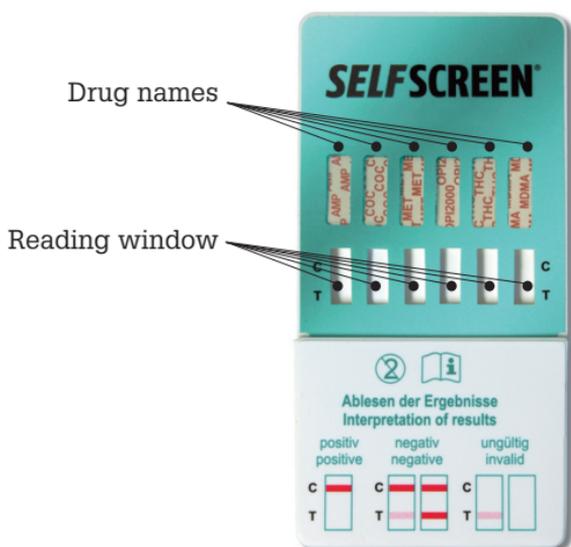

# Reading Test Results



1

**AMP positive**  
i.e.  
**Amphetamines**  
were detected



2

**COC positive**  
i.e.  
**Cocaine**  
was detected



3

**MET positive**  
i.e.  
**Methamphetamine**  
was detected



4

**OPI positive**  
i.e.  
**Opiates**  
were detected



5

**THC positive**  
i.e.  
**Marijuana**  
was detected



6

**MDMA positive**  
i.e.  
**Ecstasy**  
was detected

# Reading Test Results



7

## negative test results

The C-lines and T-lines appear.  
No drugs were detected.



8

## positive test results

Only the C-lines appear.  
All drugs were detected.



9

## invalid test results

Only the T-lines appear.  
The test must be repeated.



10

## invalid test results

No lines appear.  
The test must be repeated.





Manufactured for Omega Diagnostics (Ireland) Limited  
by:

ulti med Products (Deutschland) GmbH,  
Reeshoop 1, D-22926, Ahrensburg, Germany

**The package contains:**

- Test Cassette  1  2
- Urine Sample Cup with Lid  1  2
- 1 Instruction Booklet

**You additionally need:**

- a watch/timer



Omega Diagnostics (Ireland) Limited

Unit 28A, Bullford Business Campus,  
Kilcoole,  
Co. Wicklow

Telephone: 01 287 8158  
Web: [www.omegairl.com](http://www.omegairl.com)  
Email: [info@omegairl.com](mailto:info@omegairl.com)