

First In The World


Biocard™ *Chlamydia pneumoniae* IgM Rapid Test

**A simple and reliable rapid test for the detection of IgM antibodies
against *Chlamydia pneumoniae* antigen from serum, plasma,
or fingertip blood sample
REF 3-034-000**

- Direct detection of anti-*Chlamydia pneumoniae* IgM antibodies from serum or fingertip blood sample
- Easy and quick test procedure, result in 10 minutes
- Room temperature storage, shelf life 24 months
- Sensitivity and specificity presented on other side of the paper

* CE-marked

* 10 tests per package

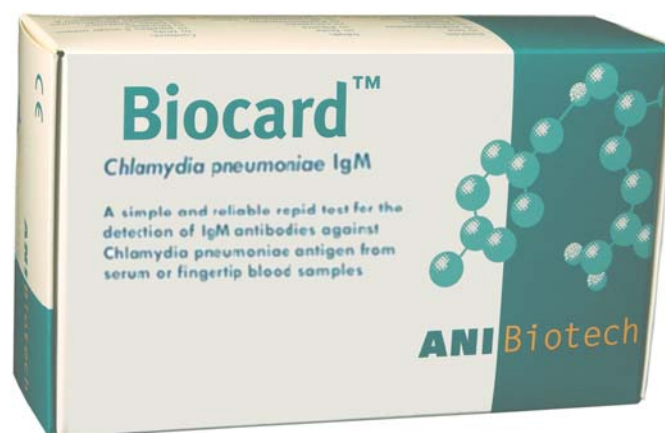

Biocard™ *Chlamydia Pneumoniae* IgM

SENSITIVITY AND SPECIFICITY:

The sensitivity and specificity of the Biocard™ *Chlamydia pneumoniae* IgM test was studied with 42 clinical samples and the results were compared to the *Chlamydia pneumoniae* IgM MIFA (Ani Labsystems Ltd.):

	<i>C. pneumoniae</i> IgM MIFA	
	—	+
Biocard™ <i>Chlamydia pneumoniae</i> IgM	— 10	+ 0
	+ 4	28

Biocard™ *Chlamydia pneumoniae* IgM test showed sensitivity of 100 % and specificity of 87.5%. Test has also been found to be negative with following serum samples:

Chlamydia trachomatis IgM
Chlamydia trachomatis IgG
Chlamydia trachomatis IgA
Bordetella pertussis IgM
 Eppstein-Barr virus IgM
 Cytomegalovirus IgM
Mycoplasma pneumoniae IgM
 Herpes simplex virus IgM
 Rubella IgM
Toxoplasma gondii IgM