

MULTI DRUG CONTROL TEST DEVICE 410

One Step Panel Test
for the detection of drugs in human urine

Urine test for evidence of:

1	Amphetamine (AMP)	cut-off: 1000 ng / mL
2	Barbiturates (BAR)	cut-off: 300 ng / mL
3	Benzodiazepines (BZD)	cut-off: 300 ng / mL
4	Cocaine (COC)	cut-off: 300 ng / mL
5	Ecstasy (MDMA)	cut-off: 500 ng / mL
6	Methamphetamine (MET)	cut-off: 1000 ng / mL
7	Morphine (MOR)	cut-off: 300 ng / mL
8	Methadone (MTD)	cut-off: 300 ng / mL
9	Tricyclic Antidepressants (TCA)	cut-off: 1000 ng / mL
10	Marijuana (THC)	cut-off: 50 ng / mL

Product ID # 008A410

1. Remove the cap to expose the absorbent tips.
2. Immerse the tips of the test into specimen until chromatography is clearly visible! Do not allow urine to contact the test device at any time.
3. Replace the cap to cover the absorbent tips and lay on a flat surface.
4. Read results within 5 to 8 minutes. Do not interpret results after more than 10 minutes.



Positive



Negative



Invalid

Negative: Two lines appear. One red or pink line appears in the control region (C), and another line in the test region (T).

Positive: One red line appears in the control region (C). No line appear in the test region (T).

Invalid: Control line fail to appear.

Note: The Multi DrugControl Test provides only a preliminary analytical result

Read
instructions
carefully
before
using
the test



2°C...30°C



Product ID # 008A410

Available in 3, 5, 6 and 10 panel versions



ONE STEP

ulti
med