

**One step test device for the qualitative detection of Benzodiazepines in human urine  
For professional in vitro diagnostic use only.**

**INTENDED USE**

The **DrugControl BENZODIAZEPINES TEST** is a lateral flow chromatographic immunoassay for the detection of Oxazepam (major metabolite) and related Benzodiazepine compounds in human urine at the cut-off concentrations shown below:

TEST DEVICE	SUBSTANCE	CAS - No	Cut Off Limit Value [ng / mL]
Benzodiazepines	Nitrazepam	[146-22-5]	100
	Clobazam	[22316-47-8]	100
	Temazepam	[846-50-4]	100
	Lorazepam glucuronide	[-]	150
	Alprazolam	[28981-97-7]	200
	Diazepam	[439-14-5]	200
	Clorazepate dipotassium	[57109-90-7]	200
	Norchlordiazepoxide	[-]	200
	Oxazepam	[604-75-1]	300
	Desalkylflurazepam	[2886-65-9]	300
	Flunitrazepam	[1622-62-4]	400
	Nordiazepam	[-]	400
	Chlordiazepoxide HCl	[438-41-5]	750
	Clonazepam	[1622-61-3]	800
	$\alpha$ -Hydroxy-alprazolam	[-]	1,200
	Bromazepam	[1812-30-2]	1,500
	Chlordiazepoxide	[58-25-3]	1,500
	Delorazepam	[2894-67-9]	1,500
	Lorazepam	[864-49-1]	1,500
	Estazolam	[29975-16-4]	2,500
Triazolam	[28911-01-5]	2,500	
Midazolam	[59467-70-8]	12,500	

This assay provides only a preliminary analytical test result. An alternate chemical method must be used in order to obtain a confirmed analytical result. Gas chromatography/mass spectrometry (GC/MS) or liquid chromatography/mass spectrometry (LC/MS) are the preferred confirmatory methods. Clinical consideration and professional judgement should be applied to any drug of abuse test result, particularly when preliminary positive results are used.

**REAGENTS**

The **DrugControl BENZODIAZEPINES TEST** contains anti-Benzodiazepines antibody coupled particles and Benzodiazepines-protein conjugate. The control line system contains a goat anti-mouse antibody.

**PRECAUTIONS**

- For in vitro diagnostic use only.
- Do not use after the expiration date.
- The test device should remain in the sealed pouch until use.
- Do not moisten nitrocellulose membrane with urine samples.
- Add 3 drops of specimen (min 180  $\mu$ l per assay).
- Avoid cross-contamination of urine samples by using a new specimen collection container for each urine sample
- All specimens should be considered potentially hazardous and handled in the same manner as an infectious agent
- The used test device should be discarded according to federal state and local regulations.

**STORAGE AND STABILITY**

The **DrugControl BENZODIAZEPINES TEST** can be stored at room temperature or refrigerated (2 – 30 °C). The test is stable through the expiration date printed on the sealed pouch. The test must remain in the sealed pouch until use.

- Do not freeze.
- Do not use beyond the expiration date.

**SPECIMEN COLLECTION AND PREPARATION**


The urine specimen must be collected in a clean and dry container. Urine collected at any time of the day may be used. Urine specimens exhibiting visible precipitates should be centrifuged, filtered, or allowed to settle to obtain a clear specimen for testing. Urine specimens may be stored at 2 - 8°C for up to 48 hours prior to testing. For long-term storage, specimens may be frozen and stored below -20 °C. Frozen specimens should be thawed and mixed before testing.

**DrugControl**<sup>®</sup>**BENZODIAZEPINES  
008A091****ulti  
med**<sup>TM</sup>**MATERIALS PROVIDED**

- Test device
- Dropper
- Package insert

**MATERIALS REQUIRED BUT NOT PROVIDED**


- Specimen collection container
- Timer

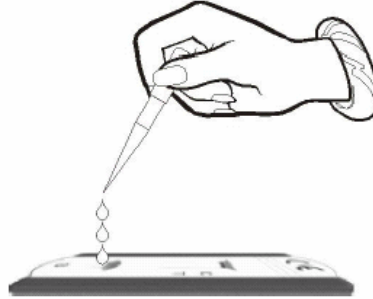

Pipette urine specimen into the sample well of **DrugControl** BENZODIAZEPINES TEST

**DIRECTIONS FOR USE**

- 1 Allow the test, urine specimen, and / or controls to reach room temperature (15 – 30 °C) prior to testing.
- 2 Bring the pouch to room temperature before opening it.
- 3 Remove the test device from the sealed pouch and use it as soon as possible.
- 4 Place the test device on a clean and level surface.
- 5 Add 3 drops of specimen (min. 180 µl) to the specimen well.
- 6 Avoid trapping air bubbles in the specimen well (S).
- 7 Place the test on a non-absorbent flat surface, start the timer and wait for the red line(s) to appear.
- 8 The result should be read at 5-8 minutes. Do not interpret after more than 10 minutes.

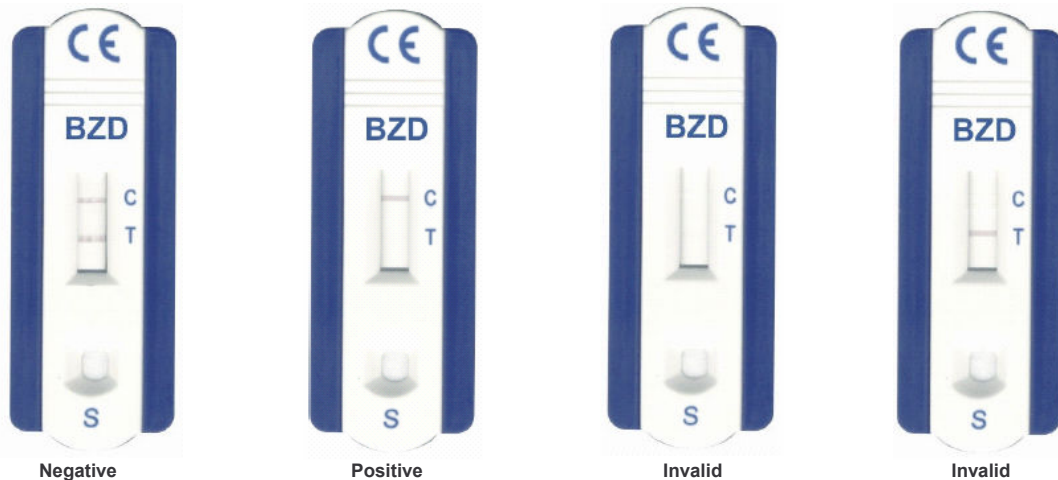**INTERPRETATION OF RESULTS**

**Negative:**\* Two lines appear. One red line should be in the control region (C), and another apparent red or pink line should be in the test region (T). This negative result indicates that the concentrations of the substances detectable with this test are below the cut-off concentration (substances & cut-off concentrations see table on page 1) or that they are not present.

**Positive:** One red line appears in the control region (C). No line appears in the test region (T). This positive result indicates that the concentration of at least one of the substances detectable with this test exceeds the cut-off concentration (substances & cut-off concentrations see table on page 1).

**Invalid:** Control line fails to appear. Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for control line failure. Review the procedure and repeat the test using a new test device. If the problem persists, discontinue using the lot immediately and contact distributor / manufacturer.

\* **Note:** The shade of red in the test line region (T) may vary, but it should be considered negative whenever there is even a faint pink line.

ulti med Products (Deutschland) GmbH  
Reeshoop 1 • 22926 Ahrensburg • Germany  
Telefon: +49-4102 - 80090  
Fax: +49-4102 - 50082  
e-mail: info@ultimed.de

ulti med Products (Belgium) BVBA  
Honzebroekstraat 137 • 8800 Roeselare  
Phone: +32 +51 200 425  
Fax: +32 +51 200 449  
e-mail: belgium@ultimed.org

**CE**

August 2005  
/ F